



FOR MORE INFORMATION

PHONE 1-800-205-7719

ONLINE WWW.CELLATEREX.COM

TWITTER [@CELLATEREX](https://twitter.com/CELLATEREX)

CellerateRX® meeting the challenge of escalating wound care costs while providing quality outcomes.

CellerateRX® and Burns

A burn is an injury caused by heat, electricity, light, radiation, or friction. 1st and 2nd degree burns may be managed at home or in a healthcare facility. Proper management of burns is important as they are painful, can result in scarring, and can lead to serious complications.

CellerateRX gel is ideal for management of 1st and 2nd degree burns. It has been shown to occlude nerve endings which results in a reduction in wound pain.¹ Collagen has also been shown to minimize the potential for scarring.

Suggested application for burns:

1. Apply the gel to cover the burn
2. Cover with a non-stick dressing if desired. It is not necessary to use a cover dressing. As the aqueous component of the gel evaporates, it cools the site. The remaining hydrolyzed collagen film forms an occlusive barrier that protects the wound.

Re-application: (every 2-3 days or as desired)

1. Apply more gel to the burn. CellerateRX is biodegradable and does not need to be removed from the wound.



The CellerateRX® Difference in Just Two Weeks!

¹ Connie Johson, "Wound Pain Reduction with Hydrolyzed Collagen", SAWC 2008